

## GYM Swedish ladder

The GYM Swedish ladder is made of gymnastics wall bars that can be used to perform light, therapeutic exercises. A variety of possible exercises are illustrated on small wooden blocks on top of the ladder. These blocks can be flipped after each completed exercise, and when all exercises are done, the user will be rewarded with a celebratory message, 'Great job!'.



### Wooden modules | GYM Swedish ladder

Article no.	Frame colour	Size HxWxD	Mounting	KG	
SC00108	Plywood	225x82,5x15cm	wallmount	15 KG	birch plywood, solid wood



All our products are inspected, tested and certified by SGS and TÜV.



European standard EN 71 specifies safety requirements for toys. Compliance with the standard is legally required for all toys sold in the European Union.



All our products only use FSC certified wood, so eco- and environment friendly!



Our products are conform with health, safety and environmental protection standards for products sold within the EU.


**IKC | HQ EUROPE**  
**Dronten | The Netherlands**  
 +31 (0)321 38 77 30  
[info@senconcepts.com](mailto:info@senconcepts.com)  
[www.senconcepts.com](http://www.senconcepts.com)

**IKC | HQ ASIA**  
**Shanghai | China**  
 +86 (0) 21 6506 1232  
[info@senconcepts.cn](mailto:info@senconcepts.cn)  
[www.senconcepts.cn](http://www.senconcepts.cn)

Back panel 2 x




Bar Elastics 3 x

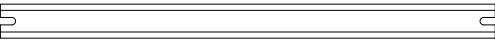


Bar 3 x



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
Bar mount 5 x



Flip board 1 x

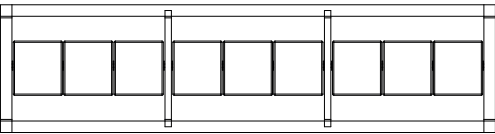


Elastics set 1 x

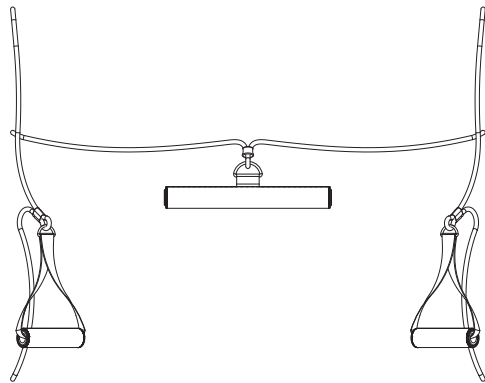

Required tools

Side Panels 2 x

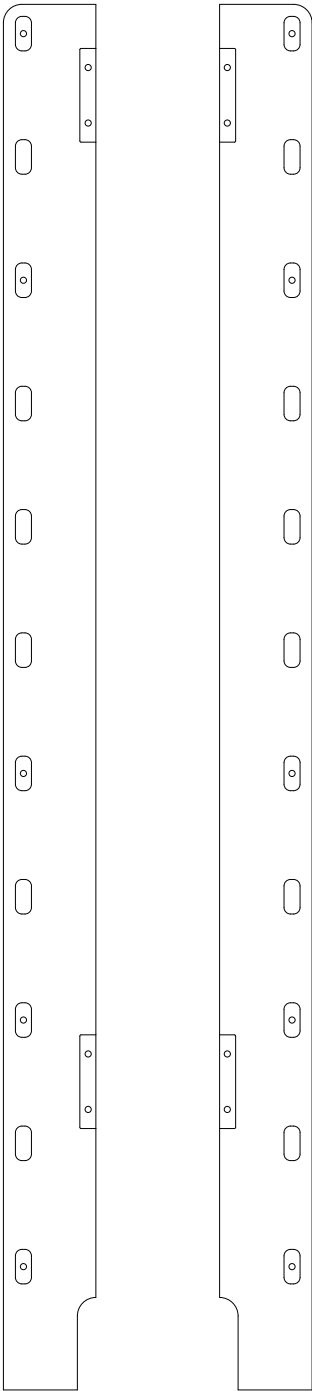

Screws

18 x

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
4 x

4 x

**Back panel** 2 x




**Bar mount** 5 x




**Side panel** 1 x




9 x

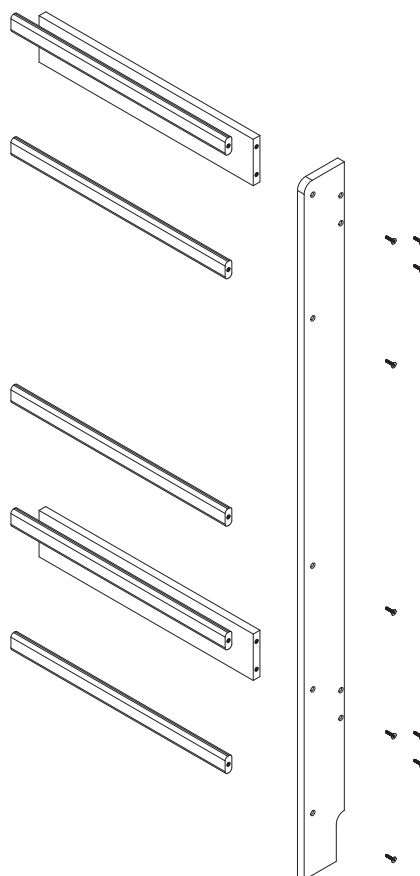



**Bar Elastics** 3 x

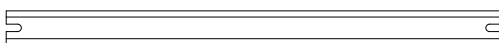

**Bar** 3 x

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

**Elastics set** 1 x

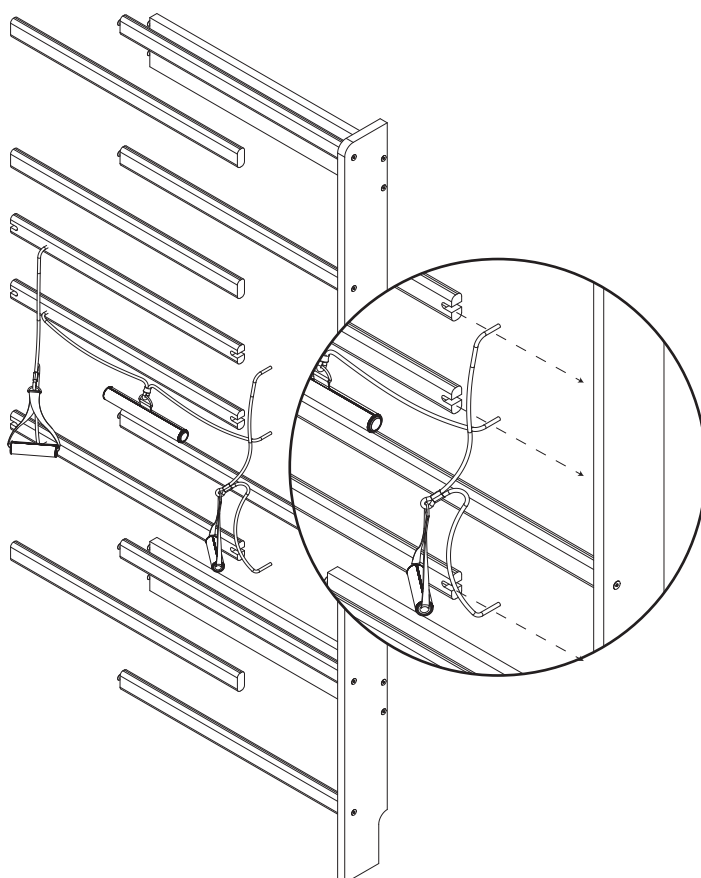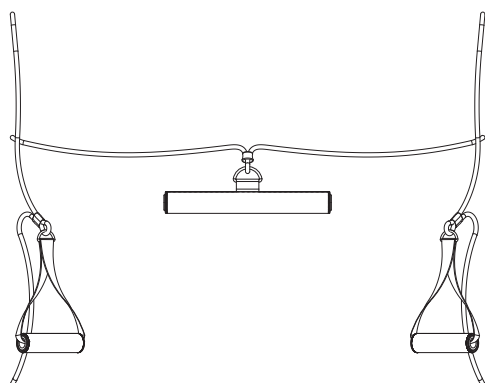


## Side panel

1 x




9 x

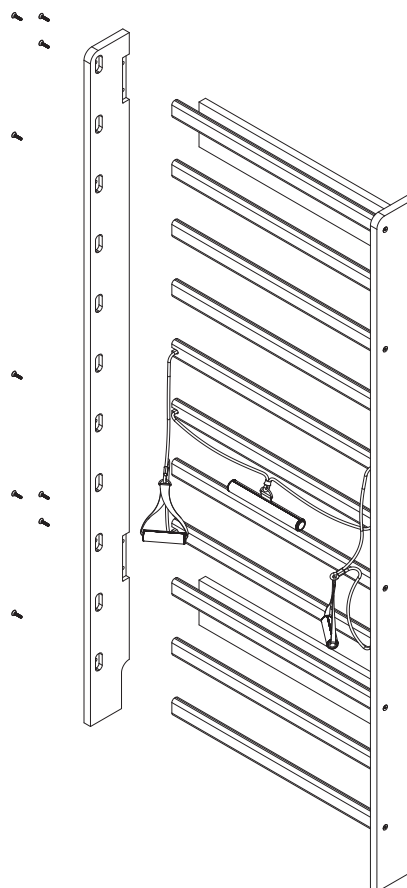



## Flip board

1 x

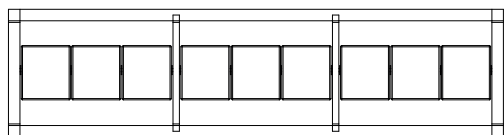

4 x

Slide in where desired. If facing the backplate is the desired location, do this step before the previous one. Located on the image is the advised location.

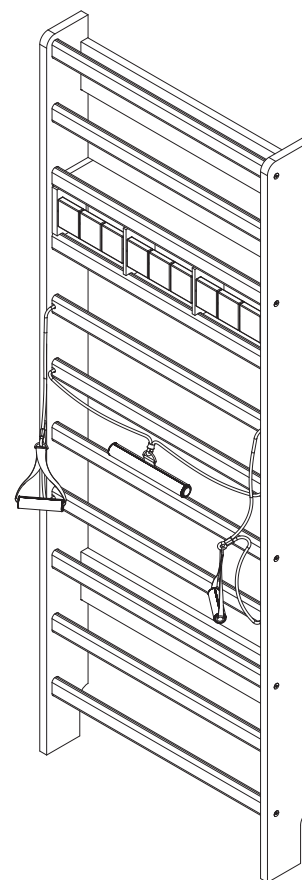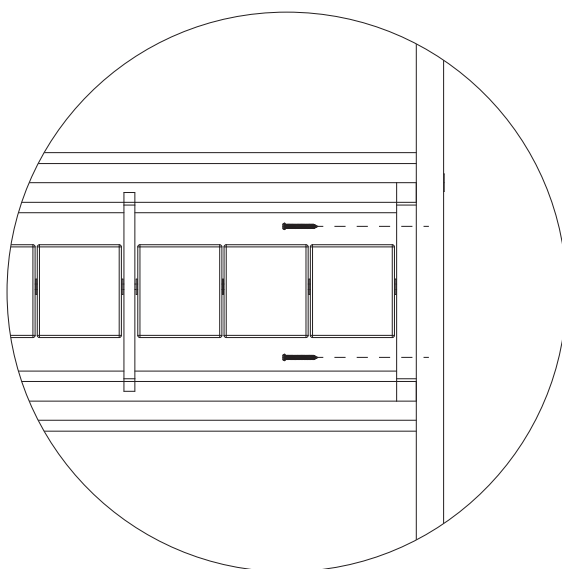

4 x

Position the Swedish ladder on a flat floor/surface directly against the wall. Make sure the ladder stands leveled and parallel to the wall. Use the supplied screws to fix the ladder to the wall.

